BAAT Medical Products
Building A04
F. Hazemeijerstraat 800
7555 RJ Hengelo
The Netherlands

Phone: +31 (0)88 565 66 00

www.baatmedical.com

3042-2

ENGLISH

Important Medical Information

PRE-OPERATIVE PRECAUTIONS

This package insert includes information on basic precautions and instructions for single use sterile medical devices from BAAT Medical Products. Please consult the applicable surgical technique for intended use, indications, contraindications, selection and use of a device and check the full labeling for other necessary information. Surgical technique brochures may be requested from the distributor or from BAAT Medical Products directly. Those using brochures published more than two years before the surgical intervention are advised to obtain an updated version.

BAAT Medical Products devices can only be used by surgeons who are fully familiar with the surgical technique required and who have been trained to this end. The operating surgeon must take care not to use the specialized instruments to exert inappropriate stress on the patient or the implants and must scrupulously comply with any operating procedure described in the surgical technique provided. For example, the forces exerted when repositioning an instrument in-situ must not be excessive as this is likely to cause injury to the patient.

To reduce the risks of breakage, care must be taken not to distort the implants or nick, hit or score them with the instruments unless otherwise specified by the applicable surgical technique. Extreme care must be taken when the devices are used near vital organs, nerves or vessels.

CONTACT

See brochure for telephone and address of local representative/distributor, or contact BAAT Medical Products.

STERILITY

Devices which are delivered sterile have been exposed to a minimum of 25kGy of gamma radiation from a cobalt 60 source.

To reduce the risk of infection the packaging of all sterile devices must be inspected for flaws in the sterile barrier or expiration of shelf life before opening. In the presence of such a flaw or expiration of shelf life, the device is considered non-sterile, and must be discarded.

Do not resterilize. Devices not labeled as sterile are non-sterile.

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