

Hygiene Instruction for Handling Medical Devices

Implants and instruments that are used on patients for clinical applications must not be used, either individually or in a set, for training purposes in public spaces (offices, conference rooms etc.) outside the hygienic area.

There is a risk of uncontrolled contamination!

Under these circumstances, professional processing may not guarantee the product cleanliness requirements, even if the validated processes indicated in the instructions for use (manual cleaning, machine cleaning with manual pre-cleaning, thermal disinfection, steam sterilization) are used.

Medical devices packaged in a sterile manner in the sterilization container must always be taken directly from the CSSD (Central Sterile Supply Department) to the operating room via the shortest route to ensure that sterility is maintained.


There is a risk of loss of sterility!

 D30003, D30008, D30009, D20301, D30087, D30088

 D30000, D30001, D30015, D30038, D30041, D30048,
D30049, D30079, D30080

D30169.a.EN
11.02.2019

 **Silony Medical Ltd.**
5 The Beacons
Hatfield
Herts, AL10 8RS, UK
Tel +44 (0)1707 32 84 77
Fax +44 (0)1707 37 60 73

 **Silony Medical GmbH**
Leinfelder Straße 60
70771 Leinfelden-Echterdingen, Germany
Tel +49 711 78 25 25 0
Fax +49 711 78 25 25 11

www.silony-medical.com

MADE IN GERMANY