

Feeling the pedicle

VI-1043
VERTICALE Pedicle Feeler,
2,3 mm



VI-1048
VERTICALE Pedicle Feeler,
2,8 mm



The VERTICALE Pedicle Feeler can be used to check the prepared screw channel for possible perforations (Fig. 1). Two Pedicle Feelers with a diameter of 2,3 mm and 2,8 mm respectively are available.

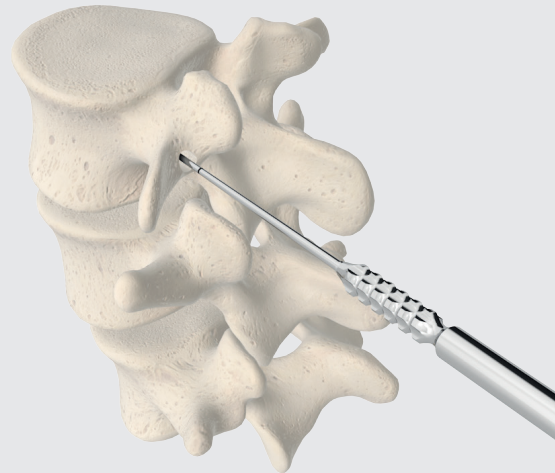


Fig. 1 Feeling the pedicle with the Pedicle Feeler

 D30003

 D30000

D30120.a.EN
08.08.2017

 **Silony Medical Europe GmbH**

Bahnhofstraße 1
28195 Bremen

Tel +49 421 246 956 0

Fax +49 421 246 956 55

 **Silony Medical GmbH**

Leinfelder Straße 60
70771 Leinfelden-Echterdingen

Tel +49 711 78 25 25 0

Fax +49 711 78 25 25 11

www.silony-medical.com

MADE IN GERMANY