

Opening the Pedicle with Starter and Tap

SI-0039
SPECIALS Awl Tap 4.5mm

SI-0040
SPECIALS Awl Starter

The desired screw insertion point into the pedicle is defined by means of anatomical landmarks and under radiographic control and the cortex subsequently opened with the Awl Starter (Fig. 1).

Very hard bone structures (e.g. sclerotic bone) may make it necessary to pre-tap the thread. For the screws with a diameter of 4.5, the SPECIALS Awl Tap 4.5mm can be used (Fig. 2).

Both instruments have a 1/4" clutch for suitable handles.



Fig.1 Opening the pedicle with the SPECIALS Awl Starter



Fig. 2 Tapping with SPECIALS Awl Tap 4.5mm

 D30003

 D30000

D30193.b.EN
10.05.2019

Silony Medical Ltd.

58A Bridge Road East
Welwyn Garden City
Herts, AL7 1JU
Phone +44 1707 32 84 77
Fax +44 1707 37 60 73

www.silony-medical.com

Silony Medical GmbH

Leinfelder Straße 60
70771 Leinfelden-Echterdingen
Germany
Phone +49 711 78 25 25 0
Fax +49 711 78 25 25 11

MADE IN GERMANY