

## Reduction with the reduction instrument

SI-0043  
SPECIALS Quick Reduction  
Wrench Torx



VI-1360  
VERTICALE Reduction Instrument



VI-1361  
VERTICALE Reduction Insert



The VERTICALE Reduction Instrument is first form-lockingly placed on the screw head. The VERTICALE Reduction Insert is inserted into the reduction instrument while making sure that the movement is not restricted. The SPECIAL Instrument is attached to the existing torx of the VERTICALE Reduction Insert and the rod is introduced into the screw head by turning the cross handle (Fig. 1). At the same time, the position of the vertebral body is corrected to posterior.

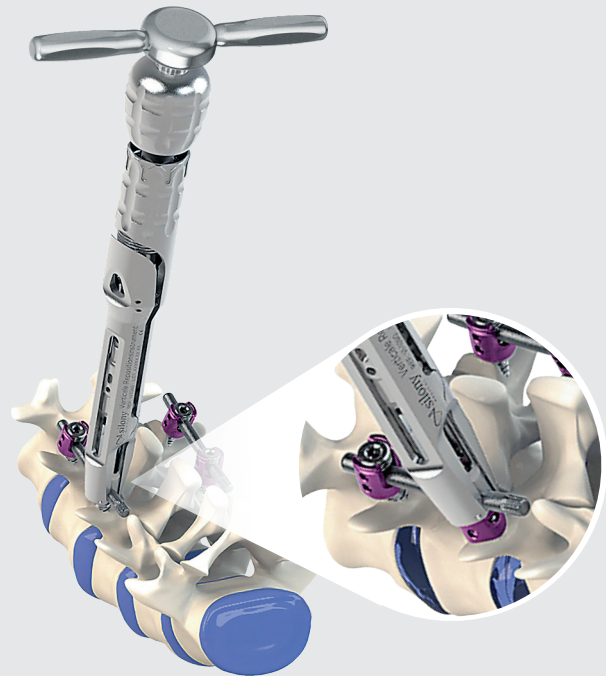


Fig. 1 Reduction maneuver with the reduction instrument

 D30003

 D30000

D30200.a.EN  
21.08.2019

 **Silony Medical Europe GmbH**

Bahnhofstraße 1  
28195 Bremen · Germany  
Tel +49 421 246 956 0  
Fax +49 421 246 956 55

 **Silony Medical GmbH**

Leinfelder Straße 60  
70771 Leinfelden-Echterdingen · Germany  
Tel +49 711 78 25 25 0  
Fax +49 711 78 25 25 11

[www.silony-medical.com](http://www.silony-medical.com)

MADE IN GERMANY