

Reduction with the reduction instrument

SI-0043
SPECIALS Quick Reduction
Wrench Torx



VI-1360
VERTICALE Reduction Instrument



VI-1361
VERTICALE Reduction Insert



The VERTICALE Reduction Instrument is first form-lockingly placed on the screw head. The VERTICALE Reduction Insert is inserted into the reduction instrument while making sure that the movement is not restricted. The SPECIAL Instrument is attached to the existing torx of the VERTICALE Reduction Insert and the rod is introduced into the screw head by turning the cross handle (Fig. 1).

At the same time, the position of the vertebral body is corrected to posterior.

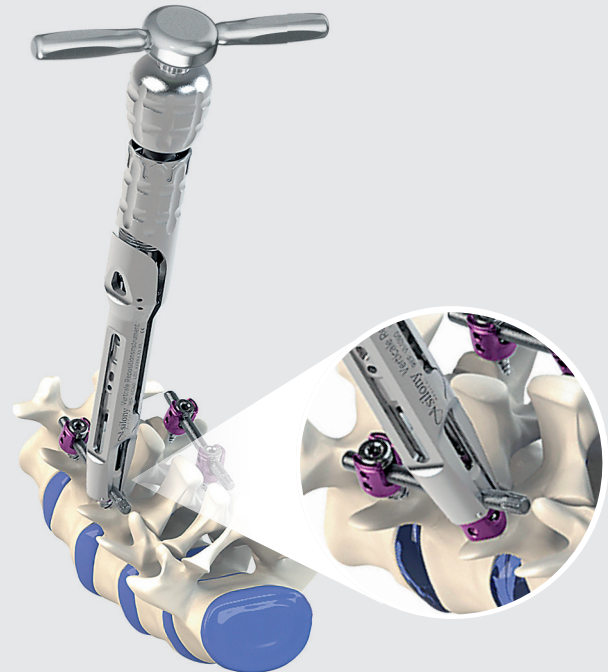


Fig. 1 Reduction maneuver with the reduction instrument


 D60007

 D60005

D60047.a.EN
21.08.2019

 **Silony Medical Corp.**
8200 NW 27th Street, Suite 104
Doral, FL 33122, USA
Phone +1 305 916 0016

US REP

 **Silony Medical GmbH**
Leinfelder Straße 60
70771 Leinfelden-Echterdingen · Germany
Tel +49 711 78 25 25 0
Fax +49 711 78 25 25 11

www.silony-medical.com

MADE IN GERMANY