

Instruction for Use (IFU047)

ACTILIF® C FLX Spinal Implants



Important Information – please read before use!

Table of Contents

1	Device Description	2
2	Indications for use	2
3	Contraindications	2
4	Warnings and Precautions	2
5	Potential adverse events.....	3
6	Packaging, storage and sterility.....	3
7	Notes for application	3
8	Magnetic Resonance Imaging (MRI) Safety Information	4
9	Labeling and Symbols.....	4

This Instruction for use covers implants of the ACTILIF C FLX systems.

Other sets of manufacturer's instructions are available containing information required for application of the systems:

- Reprocessing Instructions for Instruments (LBL379)
- Surgical Technique Guide (STG025)

The system-specific product information can also be viewed at the following links:

- <https://elabeling.silony-medical.com>
- www.silonyspine.com

Any manufacturer liability is excluded in the event of non-compliance with the manufacturer's product information.

Prior to using a product distributed by Silony Spine, the surgeon is expected to carefully study the following indications/contraindications, recommendations, warnings, and advice as well as the product specific information. Silony Spine also recommends participation in appropriate user training.

Users and/or patients are required to report any (serious) incidents related to the device to the manufacturer and to the state competent authority.

Caution: Federal Law (USA) restricts these devices to sale by or on the order of a physician.

1 Device Description

The ACTILIF C FLX is a cervical intervertebral body fusion device manufactured from titanium alloy (Ti6Al4V) per ASTM standard F3001 via additive manufacturing. FLX® devices are made with a 3D-printed osteoconductive porous titanium trabecular scaffold and feature a combination of solid and porous, radiolucent FUSE-THRU® titanium sections for reduced mechanical stiffness and improved visibility compared to solid titanium implants.

The graft containment cavity is filled with bone graft (allograft and/or autograft) material. The ACTILIF C FLX interbody fusion system consists of varying heights, sagittal profiles, and footprint options to accommodate individual pathology and anatomical conditions.

2 Indications for use

The ACTILIF C FLX is intended for spinal fusion procedures at multiple contiguous levels from the C2-C3 disc to the C7-T1 disc in skeletally mature patients with degenerative disc disease (defined as back pain of discogenic origin with degeneration of the disc confirmed by history and radiographic studies) of the cervical spine. The ACTILIF C FLX is to be used in a patient who has had six weeks of non-operative treatment prior to implantation of the cage. Implants are intended to be packed with bone graft (autograft bone and/or allogenic bone graft composed of cancellous and/or corticocancellous bone graft). Patients with previous non-fusion spinal surgery at the treated level may be treated. The ACTILIF C FLX is intended to be used with a supplemental fixation system.

3 Contraindications

- Osteoporosis, sepsis
- Infection or inflammation at or near the operative site
- Fever of undetermined origin
- Allergy to implant materials
- Patient is unable or unwilling to follow post-operative instructions
- Disease or condition which precludes the possibility of healing
- Prior fusion at the level to be treated
- Any conditions not described in the indications

4 Warnings and Precautions

- Patients with previous spinal surgery at the levels to be treated may not experience the same clinical outcomes as those without a previous surgery.
- Selection of an appropriately sized device for the patient is important and increases the likelihood of a satisfactory outcome.
- The implantation of the intervertebral body fusion device should be performed only by experienced spinal surgeons with specific training in the use of this type of device.
- Do not use if the package is damaged or opened. Contents may not be sterile.
- Do not use if current date exceeds label expiry date.
- Do not re-sterilize sterile implants.
- Instrumentation provided with the implants must be used in accordance with the approved surgical technique.
- Do not use excessive force when introducing and positioning the implant within the intervertebral body space to avoid damaging the implant.
- Re-usable surgical instruments must be re-sterilized prior to next use.
- Do not reuse the device even if the device shows no external signs of damage. Internal stresses from previous use may cause early failure.
- Should not be used with components of any other system or manufacturer.
- Based on fatigue testing results, when using the ACTILIF C FLX system, the physician/surgeon should consider the levels of implantation, patient weight, patient activity level, other patient conditions, etc., which may have an impact on the performance of this system.

5 Potential adverse events

Potential risks or adverse effects identified with the use of this intervertebral body fusion device, which may require additional surgery are similar to those of other spinal systems, and include, but are not limited to:

- Dysphagia or dysphonia.
- Bending or breakage of the components
- Foreign body (allergic) reaction
- Infection
- Pseudoarthrosis (i.e., non-union)
- Bone loss due to resorption or stress shielding
- Loss of neurological function
- Neurological difficulties such as radiculopathy, paresthesia, new or continued pain, numbness/tingling, neuroma, dural tears, neuropathy and neurologic deficit
- Loss or impairment of bowel, sexual, and/or bladder function
- Vascular damage resulting in excessive blood loss
- Spinal cord impingement or damage
- Fracture, damage, degenerative changes or instability of any bone above and/or below the level of surgery
- Additional surgery
- Death

6 Packaging, storage and sterility

6.1 Packaging

- Packaging of the components should be intact upon receipt. Damaged packages or products should not be used and should be returned to Silony Spine.

6.2 Storage

- The ACTILIF C FLX device can be shipped and stored at ambient conditions.

6.3 Sterility

- All components of the ACTILIF C FLX device are provided sterile by gamma irradiation for single use only.
- System instrumentation must be sterilized as indicated in instruction for use for cleaning and sterilization (LBL379).

7 Notes for application

Use of the ACTILIF C FLX device should only be considered when the following pre-operative, intra-operative and post-operative conditions exist:

6.1 Pre-operative

- Patient meets the indication criteria described and does not have any contraindications.
- The surgeon should determine the construct prior to surgery to ensure that the required components in the necessary sizes are available.

6.2 Intra-operative

- All components are inspected and determined to be free of damage.
- Once the ACTILIF C FLX has been introduced, additional anterior or posterior instrumentation is employed if deemed appropriate by the surgeon, who should consider factors such as the stability of the spinal column after fixation and potential risk associated with a subsequent surgical procedure to remove and/or replace these surgical appliances.
- Bone graft (autograft and/or allograft) is placed in the area to be fused.

6.3 Post-operative

- The choice to administer post-operative antibiotics is at the discretion of the surgeon.

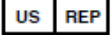
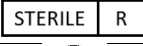
- Post-operative mobilization and rehabilitation is at the discretion of the surgeon dependent on clinical and radiological progress.
- The need for external orthotic support is not mandatory with the final choice based on surgeon preference, patient condition and intra-operative findings that might influence implant security.
- The patient is to be instructed to reduce undue stress on the implant as a precaution to avoid clinical problems that could result in fixation failure.
- The patient is to be instructed to follow the post-operative regime.

8 Magnetic Resonance Imaging (MRI) Safety Information

- Non-clinical testing demonstrated that the ACTILIF C FLX is MR Conditional. A patient with this device can be scanned safely in an MR system under the following conditions:
 - Static magnetic field of 1.5 Tesla and 3-Tesla only.
 - Maximum spatial gradient magnetic field of 2,000-Gauss/cm (20-T/m)
 - Maximum MR System reported whole body averaged specific absorption rate (SAR) of 2-W/kg for 15 minutes of scanning (i.e., per pulse sequence) in the Normal Operating Mode.
- Under the scan conditions defined, the ACTILIF C FLX implant is expected to produce a maximum temperature rise of 2.0°C after 15 minutes of continuous scanning.
- In non-clinical testing, the image artifacts caused by the ACTILIF C FLX implant extends approximately 10-mm from this implant when imaged using gradient echo pulse sequence and a 3-Tesla MR system.

9 Labeling and Symbols

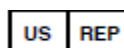
Symbols according to ISO 15223-1 and 21 CFR 801 Labeling.

Symbol	Explanatory Text
	Manufacturer
	Date of manufacture <i>Including the date in the following format: YYYY-MM-DD</i>
	US Representative
	Federal law in the USA restricts this device to sale by or on the order of a physician.
	Article number
	Lot number
	Number of items
	Medical device
	Unique device identification
	Use by date <i>Including the year and month in the following format: YYYY-MM-DD</i>
	Consult instruction for use
	Caution
	Sterilized using irradiation
	Do not reuse
	Do not resterilize
	Do not use if the packaging is damaged

	Keep dry
	Keep away from sunlight
	MR conditional (The term MR is synonymous with MRI and means magnetic resonance imaging.)
	Contact
	Silony Logo (for identification of the legal manufacturer in the course of direct marking of products)



Silony Medical GmbH
Leinfelder Straße 60
70771 Leinfelden-Echterdingen
Germany



US Representative & Distributor
Silony Spine Corp
3201 N Federal Hwy, Suite 201
Oakland Park, FL 33306
USA

Telephone: +49 (0)711-782 525 0
Fax: +49 (0)711-782 525 11
E-Mail: info.stuttgart@silony-medical.com



<https://www.silonyspine.com/contact>
www.silony-medical.com/ifu